

# cardiolife AED

Automated External Defibrillator  
AED-2152K



*Fighting Disease with Electronics*

 **NIHON KOHDEN**



# Only 3 steps to save a **life**

## Step 1. Turn on the AED

When you turn on the power, the lid opens automatically.  
Simple voice instructions guide you through the entire procedure.

## Step 2. Attach the electrode pads

AED pads are pre-connected, and there is no need to distinguish right from left.  
Just remove the backing and press them onto the patient's chest.

## Step 3. Press the shock button\*

After you attach the pads to the patient's chest, analysis starts automatically in instruction mode.  
If the AED indicates that defibrillation is needed, just push the blinking shock button. Anyone can save a life!

\*In EMS mode, the operator has to decide the ECG timing analysis by pushing the analysis button.

**Self test for reliability** The AED has a self check function with daily and monthly tests.



### System Test

Indicates current operating condition with green and red signals.



### Battery Test

Indicates remaining battery life and possible trouble points.



### Pads Test


Nihon Kohden pad connectors have special technology\* to check and indicate the pad condition.  
\*patent pending






Automated External Defibrillator  
**AED-2152K**

### User-friendly onscreen operation guide

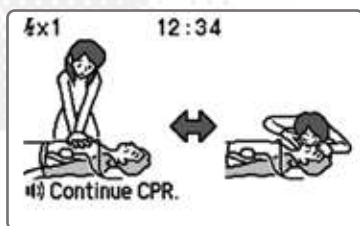

CPR instruction

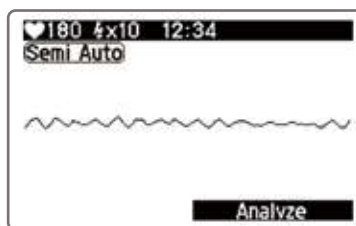

Semi auto VF analysis



AED-2152K can be used for child by changing pads and mode switch.

## Save data and review it on a PC

AED-2152K can save 90 minutes of ECG data with event annotation. You can review the data on a PC for diagnosis.



## Bluetooth interface

Rescue data and self-test results can be transferred to a PC by *Bluetooth* wireless connection.

The *Bluetooth* word mark and logo are registered trademarks owned by the Bluetooth SIG, Inc.

## Consumables

- Rechargeable Battery (SB-220V)



- Battery Pack (SB-212VK)



- Battery Pack (SB-214VK)



- Defibrillation Pads (P-530K)



## Options

- AED Box (YZ-042H8)



- AED Wall Mount Kit (KG-202V)



- Battery Charger (SB-205V)



- Carrying Bag (YC-210V)

- AED Report Viewer (QP-210VK)

## Save a life with cardiolife AED

AED-2152K (S/N 00381 and later)  
complies with RTCA/DO-160G,  
Section 21, Category M.



This brochure may be revised or replaced by Nihon Kohden at any time without notice.

Proudly distributed by :



**NIHON KOHDEN CORPORATION**  
1-31-4 Nishiochiai, Shinjuku-ku, Tokyo 161-8560, Japan  
Phone +81 (3) 5996-8036 Fax +81 (3) 5996-8100  
[www.nihonkohden.com](http://www.nihonkohden.com)

7507 CAT.No.60-042B '13.6 JPC. C.